



**Silhouette, adidas and Daniel Swarovski Paris crystal eyewear
Rimless Eyewear**

Cleaning Instructions:

Cleaning Cloth-Is used for cleaning finger prints and/or smudges only
Remove the visible dirt and grit from lens surface by lightly brushing with edge of cloth prior to cleaning. Carefully wipe the lens in a circular motion while holding the edge of the lens not the bridge. The cloth can be washed and re-used repeatedly with a mild detergent. Do not dry the cloth with a dryer sheet or water softeners because it will leave a deposit on your lens.

Regular Cleaning-Should be done once daily for heavy oil and dirt deposits.
These should be removed with warm running water and a dishwashing detergent (Ivory clear is excellent). Dry with a soft, clean cloth while holding the edge of the lens not the bridge. A lens cleaner specially formulated for Anti-reflection coatings can be used between soap and water cleanings.

Caution

When not wearing your eyewear store them in our protective case. Do not lay eyewear facedown and avoid using abrasive substances on or around the lens.