

## General glazing information::

- Only use 8 base lenses (front curve)
- In order to obtain the right eyewear, the lenses should be corrected. You can download the correction program Silhouette wrap-around calculator from our website (<http://b2b.silhouette.com>) free of charge
- Please consider 18° bow angle (geometrically defined)
- RXability (8 base) maximum -4,00/+2,00 incl. 2,00 cyl. dpt.
- In higher minus power temple contact with the lens is possible in the temporal area.  
The closure of the temple may be obstructed in the temporal area.
- Lensformers available


## Options for adjusting the eyewear to individual wearers:

### Inclination

- Make sure to fix the lens in the temporal area before inclination
- Inclination metal hinge (note: only use parallel action pliers with plastic jaws) (see figure 1)

### Temples

- Marks on the inside of the temples (5 mm spacing) make it easier to shorten the temples (see figure 2-7)
- The temples can be adjusted to the shape of the head and ears (see figure 8-9) (without heating)
- Temples are available as spare parts

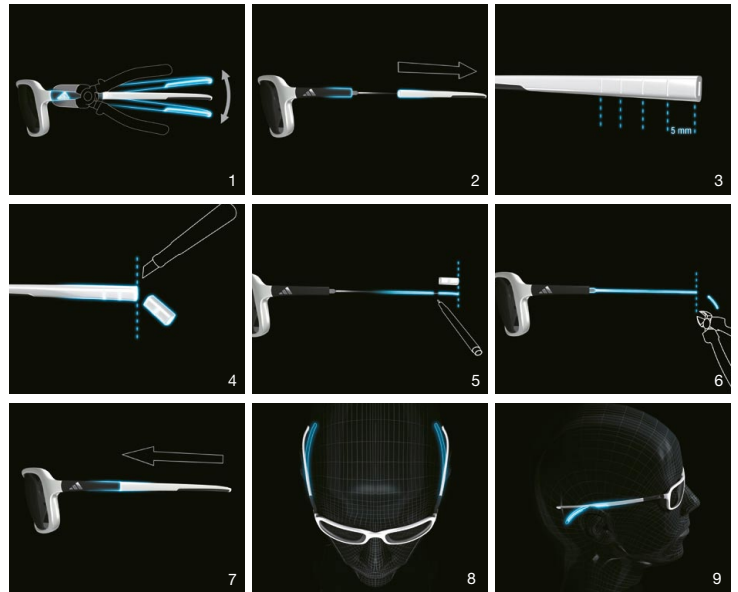

# support your vision

**Silhouette International Schmied AG**  
 adidas Global Licensee  
 Ellbogner Straße 24 / A-4021 Linz  
 Tel. +43/732/371117  
 Fax +43/732/371114  
 office@adidas-ep.com